




# Baby Cry Detector

## INSTRUCTION MANUAL

Specially designed for deaf and hearing impaired people.

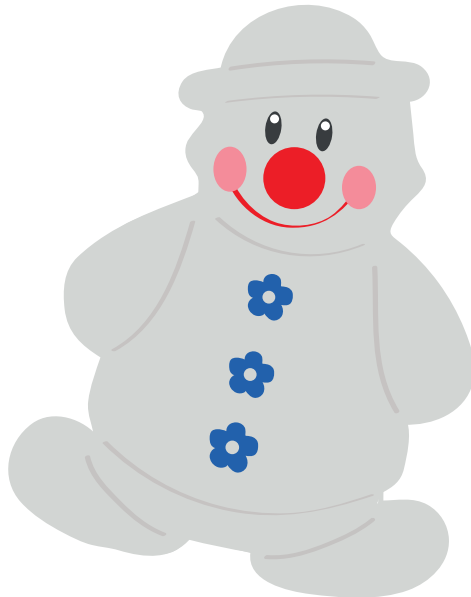




All your products sent together are already connected to each other.

# Summary :

- 4 :** The contents of your package
- 5-10 :** Baby cry detector DBC13

# The contents of your package

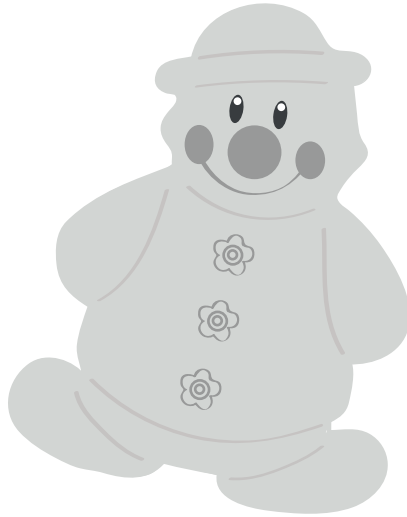


(A)



(B)

- (A) - Baby Cry detector
- (B) - Manual instruction in english

# Baby cry detector

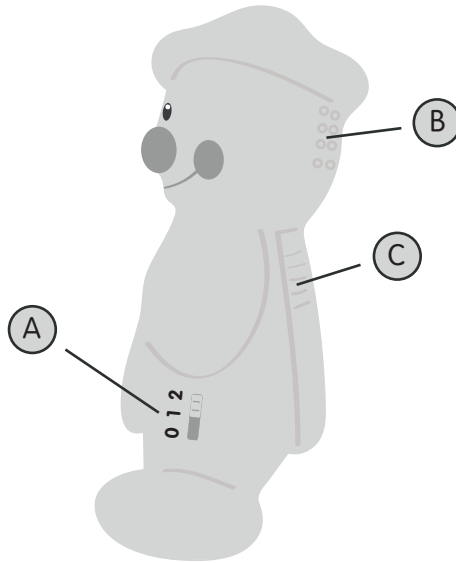

- (A) - Set button
- (B) - Audio sensor
- (C) - Battery cover

## Technical Specifications

Batteries : 2 x AAA (not included)

Dimensions (height x width x depth) : 95 x 73 x 35 mm

Color : white and decorative patens

Weight : 64 g

Transmission frequency : 433,33 Mhz

2 years guarantee

## Product features

- Device with microphone included
- The transmitter (the Baby Cry Detector) captures baby's crying and sends a sequence of flashes or vibrations on the receivers ( Jenile light or vibrating receiver )
- The transmitter (the Baby Cry Detector) is equipped with an on/off power switch to conserve battery power
- 2 settings to adjust the hearing sensitivity

## Complementary products

- Fixed flash receiver : Centrale de gestion 8 diodes - FF8I14 / Flash Plug - FP14
- Wireless flash receiver: Flash Cube - CBM14
- Vibrating receiver: Pocket vibrator - VP15

## Setting

Setting up is very easy.

Insert 2 AAA batteries (not included) by pushing the battery cover (C).


The test button remains to the "ON" position.

Position of the set button (A) :


- 0 : OFF - the Baby Cry Detector is switched off
- 1 : the Baby Cry Detector is switched on with a light sensitivity (recommended in case the Baby Cry Detector is located very close to the baby)
- 2 : the Baby Cry Detector is switched on with a strong sensitivity ( recommended in case the Baby Cry Detector is located far from the baby)

# Use

## 1) To synchronize with the Flash Cube :

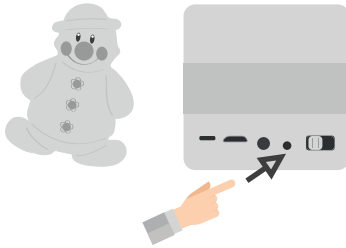


1) Switch on the Baby Cry Detector and take your Flash Cube

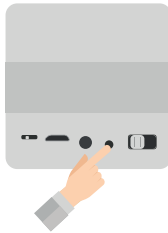

2) Press on the connection plug of your cube




3) As soon as the red light is on, release the plug button and shout in the Baby Cry Detector

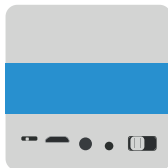

4) The Flash Cube lights up. Perfect !  
Your Baby Cry Detector is connected to your Flash Cube !

Even if you change the batteries, the connexion is memorized.  
You can relax now: you will be alerted  
as soon as your baby wakes up !

## 2) To synchronize with the pocket vibrator :

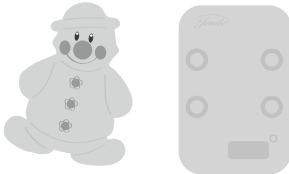


1 ) Switch on the Baby Cry Detector



2) Take your pocket vibrator and press on the plug button



3 ) As soon as the red light is on, release the plug button and shout in the Baby Cry Detector

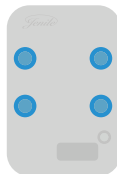


4 ) The pocket vibrator lights up

- Perfect! Your Baby Crying Detector is connected to your pocket vibrator! You can relax now: you will be alerted as soon as your baby wakes up!




## Signals and colors


The Baby Cry Detector DBC14 can be used with all the receivers from the Jenile range.

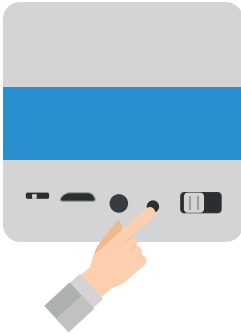

If you need to be alerted by the baby cry detector, the cube and the flash plug, will flash in blue.

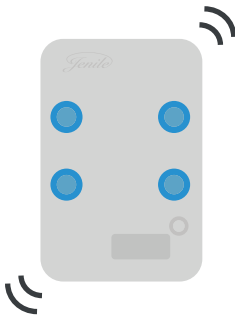

For the Baby Cry Detector, the pocket vibrator has one vibration with 4 repetitions.

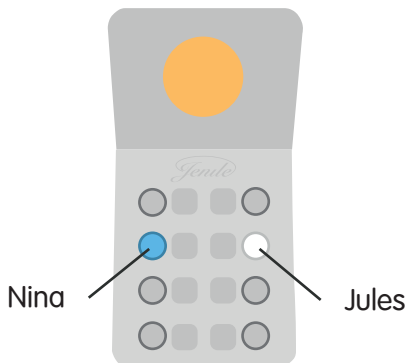

If you choose to equip yourself with several Baby Cry Detectors, the fixed flash 8 diodes will help you differentiate these alerts.



[www.jenile.com](http://www.jenile.com)

