


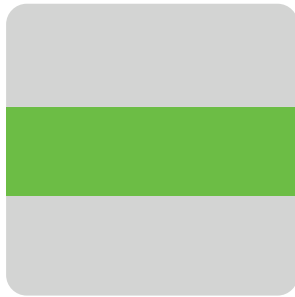

Flash cube CBM19

INSTRUCTION MANUAL

Specially designed for deaf and hearing impaired people.






All your products are already connected to each other.


Summary :

- 4 :** The contents of your package
- 5-10 :** Flash cube CBM19


The contents of your package

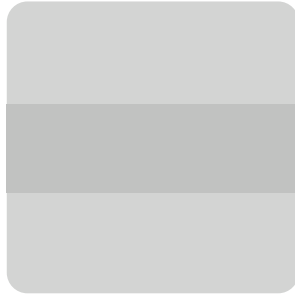

(A)

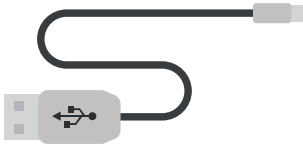


(B)

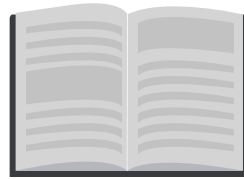

(C)

- (A) - Flash Cube
- (B) - USB connection port
- (C) - Manual instruction in english

Flash Cube CBM19

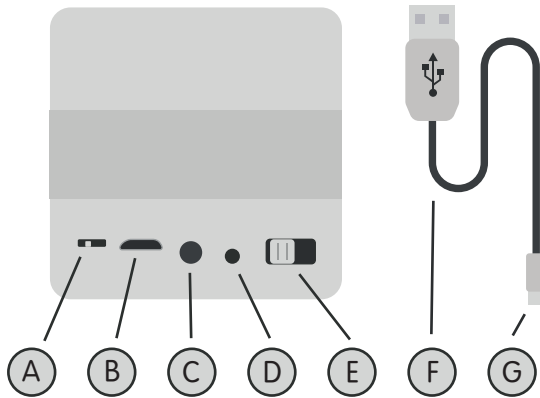

- (A) - Flashlight duration 30 / 60 / 90 second
- (B) - Micro USB charger port
- (C) - Vibrator connection port (optional)
- (D) - Connection plug
- (E) - Light / off / Light+Sound
- (F) - USB connection port
- (G) - Micro USB connection port

Technical Specifications

Wireless

Rechargeable battery

Weight: 171,5g

Distance: 200 meter maximum in free fields

Quantity of connected products: 20 units max.

Power supply: 3,7V 2200mAh Lithium battery.

Charge voltage must be limited to 4,2V

Standby current: <4,5 mA

Operating current: <800 mA

Radio frequency: 433 Mhz

Decoding format: MCU

Dimensions (height x width x depth): 9,2 x 8,8 x 4,3 cm

Autonomy: 1 month (depending of the use)


2 years guarantee (battery 1 year)

Use


Thanks to the cube, you receive the signal from the Jenile doorbell and also all the other calls, safety signals and technology Jenile. The CBM19 Jenile Flash cube includes a rechargeable battery. A LED at the bottom right of each device indicates the charge status of the battery:



Red LED: low battery, you need to recharge the cube

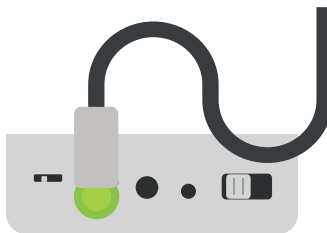

Green LED: battery is in charge

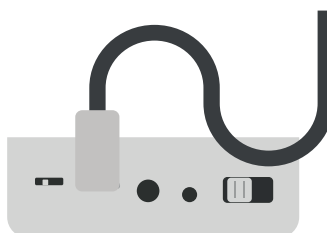


No LED lightened up:
the charge is complete

Warning: it is also possible that no LED is lightened up because the battery is completely empty. Therefore, you need to charge it.


Test

The cube is easy to test. If the cube does not work in accordance with the instruction below, we invite you to contact your seller. To test your cube press on your Jenile doorbell.

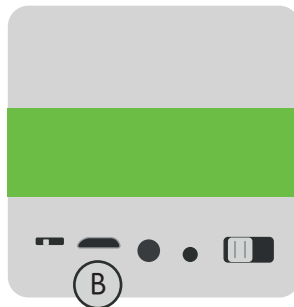

If the cube does not light up, please check that the battery is not empty by putting your cube in charge:


Insert the G port of your cable on a USB support
Insert the F port of your cable in the B plug of your device

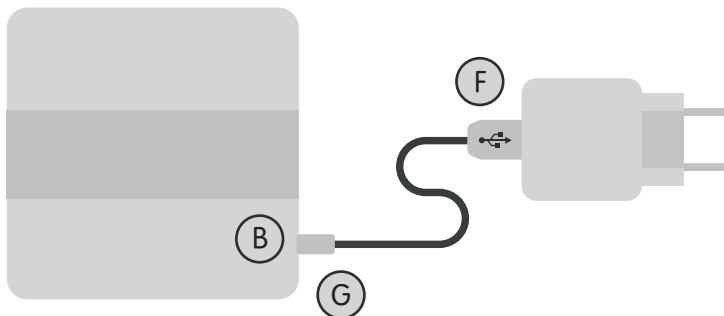


If the green LED at the bottom right of your device does not light up, it means that your device is fully charged.

Settings


if you want to complete this pack with other Jenile transmitters, then make adjustments to connect your devices between each other. To connect all your products, please follow the instructions carefully.

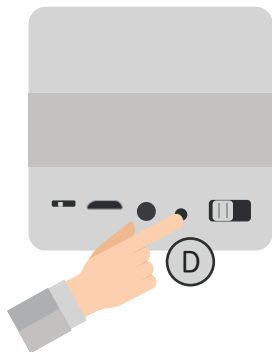

1. Press the plug button (D) on the cube.

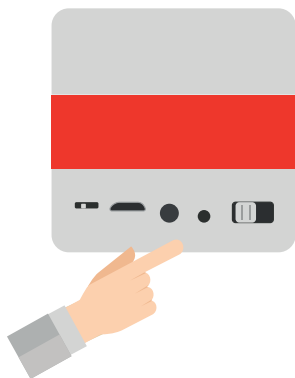

2. Release it as soon as the cube lights up in red (WARNING: do not hold down the button after the color lights up, you are at risk of deleting all previous connections).

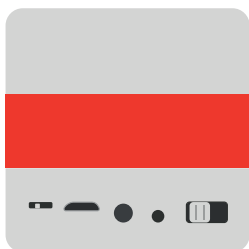


3. Immediately actuate one transmitter while the red light is on (ring the bell, press on the remote control, shout on the baby crying detector, ...) or refer to the instruction manual of your transmitter.



3 possibilities :

- Your cube flashes in all colors: congratulations! The connection is successful ! You can use all your Jenile products.
- Your cube does not flash : the connection has failed, please try again the instructions above.
- Your cube flashes 2 time in red, it doesn't flash when you put the button. The connection has failed because you pressed too long on the plug button. You erased the memory and have to restart all the connections.

The (A) button allows you to choose how long your cube flashes: 30 secondes, 1 minute or 1 minute 30.

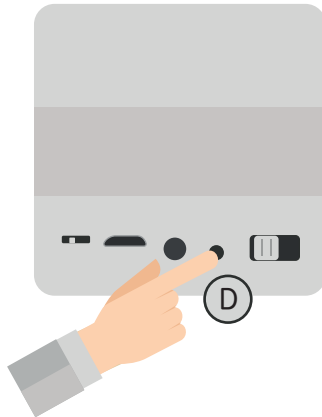


To stop the flashes, press on the D button.

Signals and colors


The CBM19 Jenile cube can be used with all the transmitters from the Jenile range.

For all alerts, the top of the cube will flash in all colors. The color at the bottom of the cube will indicate the alert type. For the doorbell, the cube turns green.

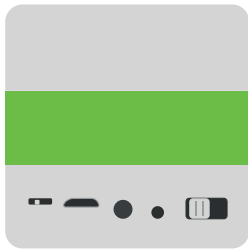


GREEN : Bells

- Door bells
- Video bells
- Video Gate
- Acoustic transmitter
- Call button, ...

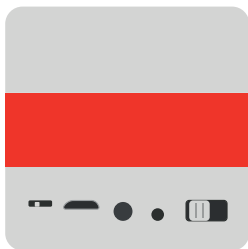


RED : Emergencies

- Smoke detector
- Gas detector,
- water flood detector, ..

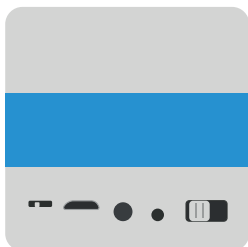

BLUE : Calls

- Text messages
- Computer call (skype, oovoo, ...), Phone, fax, intercom
- baby detectors

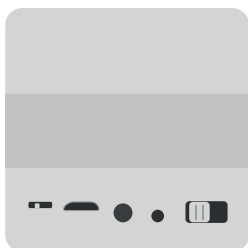

WITHOUT COLOR : Presence alerts

- Door/window opening detector
- Motion detector



www.jenile.com



CE